


# Occlusal Rim Instructions

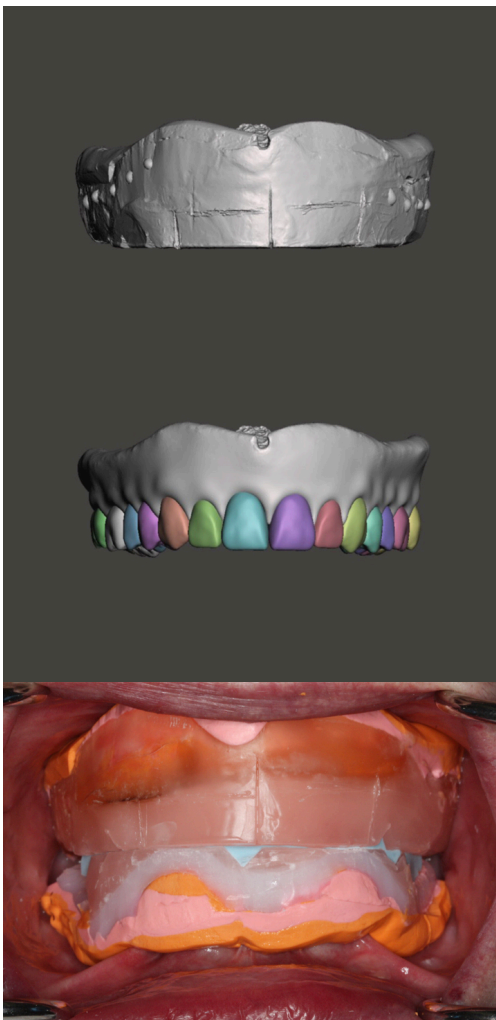

- **Shape the wax rims to develop correct:**

- Vertical dimension
- Incisal edge position
- Lip support
- Mediolateral esthetic angle

- **Mark the wax rims:**

- Midline
- Canine lines
- High smile line

Also, take a full face photo; profile; up close smile photos ( all photos taken with occlusal rim in place, should show closed mouth, repose/slightly open, smile & BIG smile).



[3dprecisiondigital.com](http://3dprecisiondigital.com)



**PRECISION**  
DENTAL ARTS

[precision-dental-arts.com](http://precision-dental-arts.com)